


# COVID-19 Rental Package: Eleganza 5 & Dolphin Therapy

medstrom<sup>+</sup>

Supporting all aspects of the critical care patient pathway.

Improved Patient Outcomes

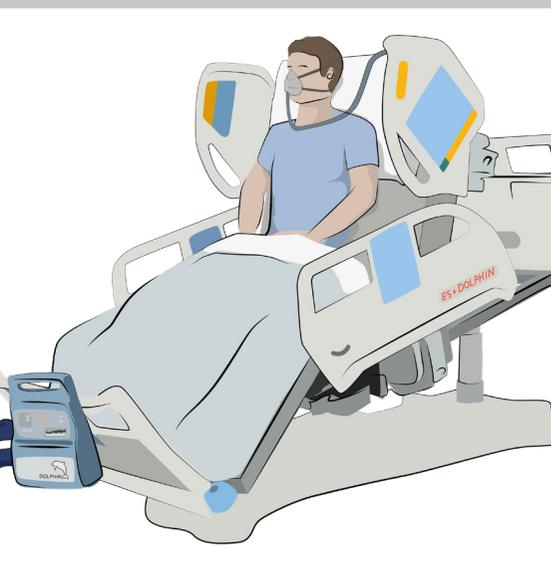

## BREATHE

When a patient enters critical care, the Eleganza 5 can be placed in to a cardiac chair position. This means the patient is physiologically upright, which aids in diaphragmatic excursion and breathing.



## SUPINE - TO - SIDE

If the patient's condition deteriorates and the prone position is required, use Eleganza 5's innovative tilting technology to eradicate manual handling and to gently turn the patient up to 15°. This facilitates an easier transition from side to prone, requiring less physical effort.



## PRONE + PROTECT

Due to the simulated fluid environment and immersive properties, Dolphin Therapy is known to reduce all skin damage, including device-related pressure ulcers, even in the most complex of patients.



## SIDE - TO - SUPINE

When the patient begins to recover, use the Eleganza 5 to move the patient from the prone position, to the side and then back to supine. Additional features, such as the Mobi-lift, can then be used to aid patient mobilisation.

UK | T: 0845 371 1717

SM691\_Feb21

E: [info@medstrom.co.uk](mailto:info@medstrom.co.uk)

IRELAND | T: 01 686 9487

E: [info@medstrom.ie](mailto:info@medstrom.ie)

For patients with more complex respiratory conditions.

## IF STEP UP IS REQUIRED

For longer term patients who may be slow weaning or have further respiratory complications, the Multicare & Dolphin Therapy is available as an enhanced bed frame that can provide Automatic Lateral turning cycles up to 30° whilst maintaining skin integrity.

Continuous lateral pulmonary rotation results in:

- **Maintenance of pulmonary blood flow to maximise oxygenation**
- **Mobilisation of chest secretions**
- **Prevention of other well-known complications of immobility**



To take advantage of our flexible and responsive rental packages, or to discuss special COVID-19 pricing, contact Medstrom 24/7/365. Our service includes clinical advice and technical support.

## UK

T: 0845 371 1717

E: [info@medstrom.co.uk](mailto:info@medstrom.co.uk)

W: [www.medstrom.com](http://www.medstrom.com)

## IRELAND

T: 01 686 9487

E: [info@medstrom.ie](mailto:info@medstrom.ie)

W: [www.medstrom.ie](http://www.medstrom.ie)